



# Ready, Steady, **EnGO!**

The wireless handpiece with file database.





Brasseler®, Komet®, Art2®, CeraBur®, CeraCut®, CeraDrill®, CeraPost®, DC1®, DIAO®, FastFile®, F360®, F6 SkyTaper®, H4MC®, KometBioSeal®, OccluShaper®, OptiPost®, PolyBur®, PrepMarker®, Procodile®, Procodile Q® et SHAX® are registered trademarks of Gebr. Brasseler GmbH & Co. KG.

ReFlex® and EndoPilot® are registered trademarks of Schlumbohm GmbH & Co. KG

Some of the products and designations mentioned in the text are trademarked, patented or copyrighted. The absence of a special reference or the sign ® should not be interpreted as the absence of legal protection.

This publication is copyrighted. All rights, also with regard to translation, reprint and reproduction (also in the form of extracts) are reserved. No part of this publication may be reproduced or reprocessed using electronic systems in any form or by any means (photocopying, microfilm or other methods) without the written permission of the editor.

Colours and products are subject to alterations.  
Printing errors excepted.

As at February 2025



# Flexible. Safe. Powerful.

EnGO – for comfort in endodontics.

With Komet's EnGO handpiece, you have a true multi-talent for endodontics in your hands. Whether reciprocating or rotating: EnGO is suited for all common types of movements and offers individual setting options for speed, torque, and angle. The specifications for your most frequently used files can be saved in the unit.

Thanks to the pre-installed parameters of all Komet systems, you are all set for your next treatment.

The wireless, ergonomic unit fits perfectly into the hand of both right-handed and left-handed users and makes work more comfortable, even during longer treatments. Thanks to its special contra-angle that can be rotated by 360°, EnGO provides optimal access to the treatment area. In addition, EnGO is particularly safe to use: the small head ensures good visibility, and the integrated apex locator allows you to prepare right up to the apex.



ENGO13

# With EnGO through endo.

The wireless handpiece with programmable file memory.

## Individual settings.

The integrated memory holds up to ten different movement programs. Speed, torque, and angle can be individually adjusted to suit each file. The parameters of all Komet systems are already pre-installed – so you are ready for your next treatment.

## Sophisticated Ergonomics.

Whether right- or left-handed: the integrated LCD display can be adjusted to your personal needs. Furthermore, EnGO fits perfectly into your hand for minimum fatigue during work. The balanced weight distribution of the unit further reduces stress to your hand and wrist.





## Optimum control during the entire treatment.

### Integrated Apex Locator.

Thanks to the integrated apex locator, your treatment becomes even safer. The automatic length measurement shows you in real time how far you have advanced towards the apex thus minimizing the risk of overpreparation.



### Optimum View.

Throughout the treatment, the narrow contra-angle head provides a particularly large field of view of the treatment area. The treatment is further facilitated by the 360° rotation of the contra-angle head which allows optimum access to the treatment site.



# That's not all yet.

These additional EnGO features round off your treatment:

- **Impressive Performance**

With up to 2,500 revolutions per minute, EnGO opens up further areas of application – while keeping its high precision.

- **Rotatable all around**

The 360° rotation of the contra-angle provides optimum access to the treatment area.

- **Automatic Movement**

For more control: EnGO starts automatically as soon as you insert the file into the root canal and stops once you remove the file from the canal.

- **Adjustable Reciprocating Movement**

The reciprocating angle can be individually adjusted in intervals of 10° to suit any customary file.

- **Easy Handling**

Through its ergonomic shape and smooth operation, EnGO ensures maximum control during treatment.

- **Wireless Charging**

Whether during work or while charging up, EnGO offers maximum flexibility thanks to its wireless design.

- **Long-lasting Battery**

Thanks to the powerful battery, you can always rely on EnGo's durable strength to last through the entire treatment - even if it takes a little longer than expected.



# Get started straight away.

Thanks to the pre-installed settings for your Komet instruments.

The parameters of all Komet systems are already pre-installed.

You can also save your own settings.

## FQ files



### FQ Opener

Operating mode CW  
Torque 2,4 Ncm ⌚<sub>opt.</sub> 300 rpm



### FQ Glider

Operating mode CW  
Torque 0,8 Ncm ⌚<sub>opt.</sub> 300 rpm



### FQ04 020 – 035

Operating mode CW  
Torque 1,8 Ncm ⌚<sub>opt.</sub> 300 rpm



### FQ04 045 – 055

Operating mode CW  
Torque 2,4 Ncm ⌚<sub>opt.</sub> 300 rpm



### FQ06 020 - 030

Operating mode CW  
Torque 2,4 Ncm ⌚<sub>opt.</sub> 300 rpm

## Procodile Q files



### Rez. Opener

Operating mode RECIP  
Torque Auto



### Procodile Q 020 – 050

Operating mode RECIP  
Torque Auto

## Other rotary files



### Opener

Operating mode CW  
Torque 2,5 Ncm ⌚<sub>opt.</sub> 300 rpm



### PathGlider

Operating mode CW  
Torque 0,5 Ncm ⌚<sub>opt.</sub> 300 rpm



### F360

Operating mode CW  
Torque 1,8 Ncm ⌚<sub>opt.</sub> 300 rpm

## Retreatment file



### Endo ReStart

Operating mode CW  
Torque 1,8 Ncm ⌚<sub>opt.</sub> 300 rpm

**Komet Dental**

Gebr. Brasseler GmbH & Co. KG  
Trophagener Weg 25 · 32657 Lemgo  
Postfach 160, 32631 Lemgo · Germany  
Telefon +49 (0) 5261 701-0  
hello@kometdental.com  
Contact for dealers:  
export@kometdental.de

**Verkauf Deutschland:**

Telefon +49 (0) 5261 701-700  
Telefax +49 (0) 5261 701-289  
info@kometdental.de  
www.kometdental.de

**Export:**

Telefon +49 (0) 5261 701-0  
Telefax +49 (0) 5261 701-329  
export@kometdental.de  
www.kometdental.com

**Komet Austria** Handelsagentur GmbH

Hellbrunner Straße 15 · 5020 Salzburg  
Telefon +43 (0) 662 829-434  
Telefax +43 (0) 662 829-435  
info@kometdental.at  
www.kometdental.at

**Komet France**

18 rue · Fourcroy · 75017 Paris  
Phone +33 (0) 1 43 48 89 90  
info@komet.fr  
www.komet.fr

**Komet Italy**

Via Gianbattista Morgagni 36,  
37 135 Verona  
Phone +39 (0) 45 11171911  
info@komet.it  
www.komet.it

**Komet UK**

Invoice an general correspondance:  
27 Old Gloucester St,  
Holborn, London WC1N 3AX  
info@kometuk.com  
www.kometuk.com