Komet Dental

Your catalog. Your selection.

Endodontics.

2025/2026

Brasseler®, Komet®, Art2®, CeraBur®, CeraCut®, CeraDrill®, CeraPost®, DC1®, DIAO®, EnDrive®, EnFill®, EnPack®, FastFile®, F360®, F6 SkyTaper®, FQ®, H4MC®, Komet BioSeal®, OccluShaper®, OptiPost®, PolyBur®, PrepMarker®, Procodile®, Rocky®, SHAX® sind eingetragene Marken der Gebr. Brasseler GmbH & Co. KG.

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Brasseler®, Komet®, Art2®, CeraBur®, CeraCut®, CeraDrill®, CeraPost®, DC1®, DIAO®, EnDrive®, EnFill®, EnPack®, FastFile®, F360®, F6 SkyTaper®, FQ®, H4MC®, Komet BioSeal®, OccluShaper®, OptiPost®, PolyBur®, PrepMarker®, Procodile®, Rocky®, SHAX® are registered trademarks of Gebr. Brasseler GmbH & Co. KG.

Some of the products and designations mentioned in the text are trademarked, patented or copyrighted. The absence of a special reference or the sign ® should not be interpreted as the absence of legal protection.

This publication is copyrighted. All rights, also with regard to translation, reprint and reproduction (also in the form of extracts) are reserved. No part of this publication may be reproduced or reprocessed using electronic systems in any form or by any means (photocopying, microfilm or other methods) without the written permission of the editor.

Colours and products are subject to alterations. Printing errors excepted.

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65 - 66 Guttaperchaspitzen
Guttapercha points

66 - 67 Papierspitzen
Paper points

67 Guttapercha Cutter
Guttapercha cutter

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74 - 77 Endo Rescue Kit
Endo Rescue Kit

78 - 81 Postendo
Post Endo

80 - 81 Composite System
Composite System

82 - 89 Schall
Sonic tips

83 - 87 SonicLine
SonicLine

90 - 107 Zubehör
Auxiliaries

91 - 92 Inserttray
Insert tray

93 - 94 Waschbox
Wash box

Endodontie

Endodontics

The systematic concept is reflected in our full range of endodontic products. Perfectly matched high-quality products allow safe, efficient and comfortable work.

Our range includes instruments for access preparation as well as manual files and the heat-treated NiTi file systems FQ and Procodile Q for the mechanical preparation of root canals. The EnDrive, which unites an endodontic motor and apex locator all in one, is designed for torque-limited operation in the root canal.

A leakproof root filling can be achieved with Komet BioSeal, a bioceramic sealer, EasySeal, a filling material based on epoxy resin, gutta-percha points or F360 Fill.

To round off our range, we offer handy accessories, e.g. interim supports, Endo Sequencer and the EndoRescue kit for removing instrument fragments. Different root posts and a composite system are available for post-endodontic treatments.

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Access cavity

The creation of a perfect access cavity is the first step of any endodontic treatment. In many cases, this is more complicated than the subsequent preparation of the root canal. The formation of the access cavity is divided into two stages: the preparation of the primary cavity (access to pulp chamber) and the secondary cavity (access to the root canal system). The success of the endodontic treatment largely depends on these two vital first steps - the creation of sufficient space and unobstructed vision. Komet offers a comprehensive range of special instruments for this.

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EndoGuard. The course is set for Endo success

The creation of a perfect access cavity is the foundation of any successful endodontic treatment.

Used immediately after opening the pulp chamber, the EndoGuard helps to carry out this important treatment step with optimum efficiency and in perfect safety.

- Removal of dentin overhangs for improved vision and easier detection of the canal entrances
- The straight access to the root canal minimizes the risk of inadvertent canal transportation and fracture of the file
- Smooth, non-cutting tip to protect the floor of the pulp chamber
- Tapered shape to prevent the preparation of undercuts
- Efficient cross-cut toothing for perfectly controlled, yet smooth work with minimum vibration

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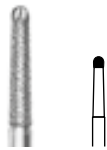
EndoGuard with safe end, with cross cut

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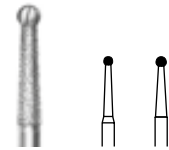
Combination bur with diamond coated working part and carbide tip, for lateral substance removal when preparing an access cavity

15802

		5
Größe · Size	Ø 1/10 mm	014
L	mm	10,0

FG · Friction Grip (FG)

15802.314. ...

014

⊖_{max} 160000 min⁻¹/rpm

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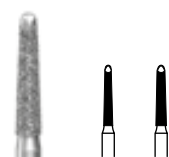
For creation of an endo access cavity

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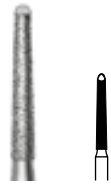
EndoTracer

EndoTracer

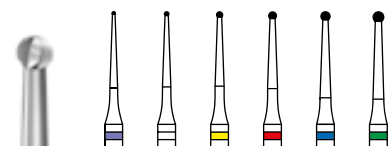
The EndoTracer is an endodontic instrument specially intended for the preparation of the endodontic access cavity, notably for the preparation of isthmuses.

- Thanks to their special design, the petite round burs - especially the sizes 004 and 006 - are ideally suitable for the fine shaping of isthmuses and canal entrances
- The EndoTracer comes in two lengths and in 6 different sizes. This means that the range comprises the perfect instrument for any clinical situation
- Provided with a particularly long, slim neck, the EndoTracer allows unobstructed view past the instrument into the access cavity
- With its total length of 34 mm and an extra 3 mm in neck region, the EndoTracer is now even more suitable for work under a microscope

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H1SML31 Gesamtlänge 31 mm

H1SML34 Gesamtlänge 34 mm

EndoTracer for the preparation of the endodontic access cavity, notably for the preparation of isthmuses

H1SML31 length 31 mm

H1SML34 length 34 mm

10

4670.205

EndoTracer Einführungsset
EndoTracer Introductory set

			
	H1SML31.205.006	1	
	H1SML31.205.008	1	
	H1SML31.205.010	1	
	H1SML31.205.012	1	
	H1SML31.205.014	1	
	H1SML34.205.006	1	
	H1SML34.205.008	1	
	H1SML34.205.010	1	
	H1SML34.205.012	1	
	H1SML34.205.014	1	

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EndoExplorer. Form folgt Funktion

EndoExplorer. Form follows function

Developed in close cooperation with the endodontic specialist Dr. Hans-Willi Herrmann, this new instrument set is designed for the ergonomic, tissue-friendly primary and secondary preparation of the endodontic access cavity.

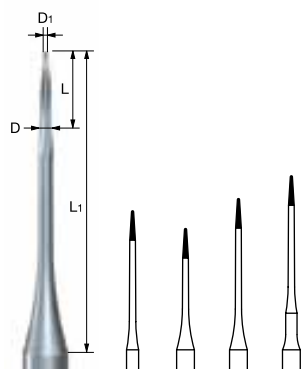
- The instrument design is optimally adapted to the requirements of microscope users
- Petite instrument head in combination with a long, slender neck for unobstructed view
- Controlled, precise work almost without pressure thanks to the sharp toothing
- Tapered instrument head to allow controlled guidance of the instrument and almost non-invasive work
- Completely made of tungsten carbide for maximum runout accuracy even after several uses

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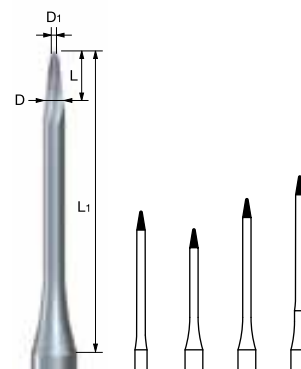
EndoExplorer for the preparation of the endodontic access cavity, tungsten carbide

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4697ST.000

Endo Access Kit Shanon Patel
Endo Access Kit Shanon Patel

	H4MCL.314.012	1
	4ZR.314.012	1
	H269QGK.314.012	1
	6801L.314.014	1
	6850.316.014	1
	8379.314.023	1
	EX1.204.007	1
	EX2.204.011	1
	H1SML31.205.006	1
	H1SML31.205.010	1

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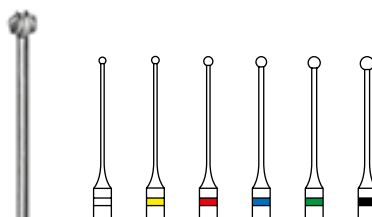
Pulp bur "Müller", stainless steel

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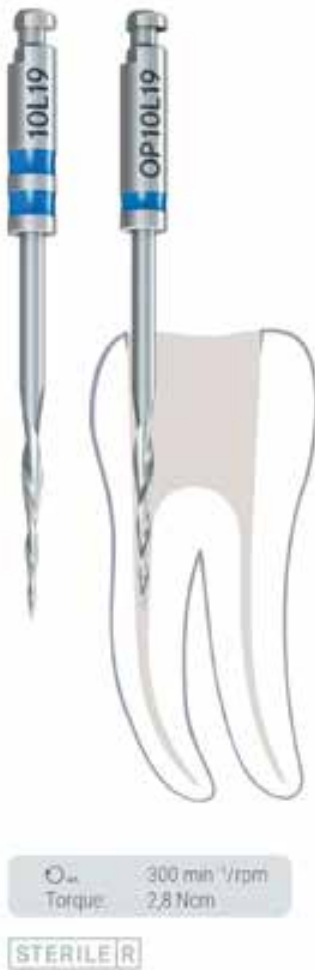
Coronal reaming

The opening of the root canal system is followed by the preparation of the root canals. Given that most bacteria are found in the coronal third of the root canal, it is particularly important to clean this section of the canal right at the start, making sure that the subsequently used files do not spread the bacteria towards the apex. This step has the added advantage of facilitating the work of the next files and improving the dentist's field of vision. Komet provides a vast line of instruments for this, from the well-established Gates-Glidden burs to openers. These files for the mechanical preparation of root canals are provided with a short working part and large taper to facilitate particularly efficient work.

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Opener

This Opener is ideally suited for the fast and generous enlargement of the canal entrance zone. It reliably removes most bacteria from the canal right at the start of the treatment. The fast-cutting design of the file ensures quick and thorough removal of the infected coronal tissue. What's more, the construction of the opener opens new perspectives: an improved view of the work field, especially in combination with magnifying glasses or a microscope. The preparatory work done by the Opener relieves the strain on the subsequently used file system and creates ideal conditions for the preparation further down the root canal.

By the way, it does not matter if you prefer working to a reciprocating or rotary method: both versions of the Opener are suitable for universal use and can be combined with any file system and preparation technique.

- Quick enlargement of the canal entrance zone
- Suitable for universal use
- Fast-cutting file design
- Less strain on the subsequently used file systems
- Improved vision
- Sterile

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Opener, taper .08, length 19 mm, sterile packed, for the straight root canal entrance area with small lumen, nickel-titanium

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Opener, taper .10, length 15 or 19 mm, sterile packed, for the straight root canal entrance area, nickel-titanium

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Reciprocating Opener, taper .08, length 19 mm, sterile packed, for the straight root canal entrance area with small lumen, nickel-titanium

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Reciprocating Opener, taper .10, length 19 mm, sterile packed, for the straight root canal entrance area, nickel-titanium

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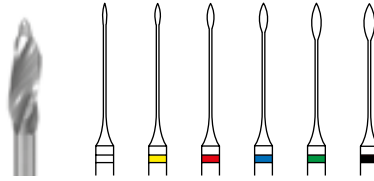
Reamer Gates Glidden "G", stainless steel

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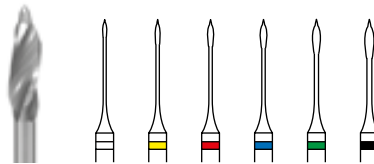
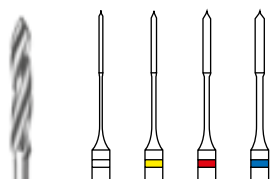
Reamer Gates Glidden "G", short, stainless steel

183L

Größe · Size	Ø 1/100 mm	070	090	110	130
--------------	------------	-----	-----	-----	-----

Winkelstück · Right-angle (RA)

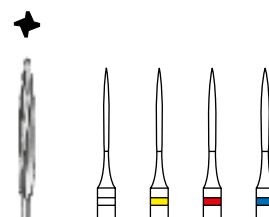
330 204 682336 ...

183L.204. ...	070	090	110	130
----------------------	-----	-----	-----	-----

○_{max} 20000 min⁻¹/rpm
Erweiterer Typ „P“, rostfreier Stahl
Reamer "P", stainless steel

182

Größe · Size	Ø 1/100 mm	090	100	120	140
--------------	------------	-----	-----	-----	-----

Winkelstück · Right-angle (RA)

330 204 680336 ...

182.204. ...	090	100	120	140
---------------------	-----	-----	-----	-----

○_{max} 20000 min⁻¹/rpm
Erweiterer Typ „B1“, rostfreier Stahl
Reamer "B1", stainless steel

Creation of a glide path

The creation of a glide path is indispensable for ensuring successful endodontic treatments. This step facilitates the correct estimation of the anatomy of the root canal, thus ensuring that the subsequently used files for root canal preparation work safely and efficiently. The glide path can be created manually or mechanically – Komet offers a complete solution for both options.

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PathGlider

PathGlider

The PathGlider is intended for the mechanical creation of a glide path. Made of highly flexible nickel titanium and provided with a non-cutting instrument tip, the PathGlider perfectly follows the course of the canal, thereby not only reducing the risk of inadvertent canal transportation and formation of undesirable steps, but also saving precious time. Thanks to its taper of .03, the instrument clears the way for the subsequently used mechanical files (e.g. F360: taper .04), making sure that they work safely and efficiently. The PathGlider can be combined with any file system. Its unique kite-shaped cross-section guarantees smooth canal walls and excellent control of the file inside the canal.

Advantages over an exclusively manual creation of a glide path:

- Reduced risk of inadvertent transportation of the canal and creation of undesirable steps
- Economy of time
- Unique kite-shaped cross-section for smooth canal walls and excellent control of the file inside the canal
- Transition taper .03 for a gradual, gentle transition to the subsequently used mechanical file systems
- Sterile

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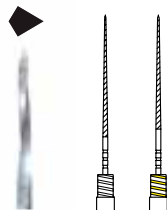
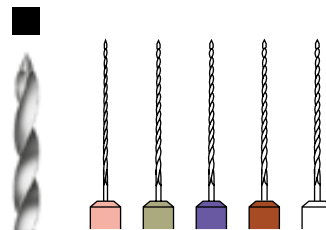
PathGlider file, taper .03, single-use file, sterile packed, for the mechanical creation of a glide path, nickel-titanium

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Manual file for probing and checking patency of root canals, stainless, heat-strengthened steel

Sortimente:
Assortments:

L = 21 mm

175 21.654.S1

2 x 006 – 010

175 25.654.S1

2 x 006 – 010

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Manual preparation

Three basic types of files are available for the manual preparation of the root canal: K-burs, K-files and Hedstroem files. The preparation of root canals is governed by the standard ISO 3630 which applies to root canal instruments. In compliance with ISO provisions, all manual files come with a taper of .02 and a toothed working part with a length of 16 mm. Beyond the requirements of the standard, manual files made by Komet offer the following additional advantages:

- Radiopaque stoppers preassembled at our works
- Clearly visible depth marks
- Ergonomic handle with knobs to prevent slipping
- Handle provided with a hole to insert a safety chain

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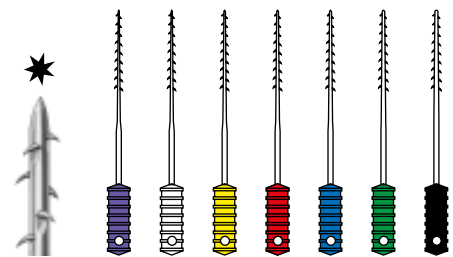
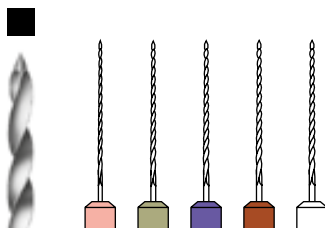
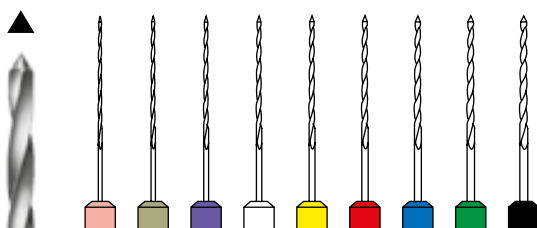
Instruments for root canal preparation, root canal reamers "K", stainless spring steel

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Instruments for root canal preparation, "K" files, stainless spring steel

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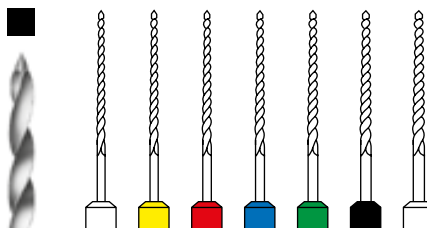
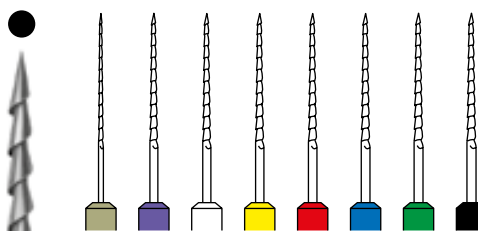
Instruments for root canal preparation, Hedstroem files „H“, stainless spring steel

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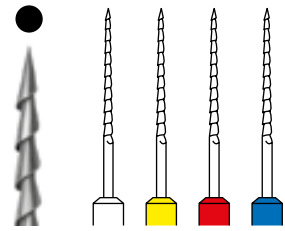
Instruments for root canal preparation, Hedstroem files "H", stainless spring steel

Mechanical preparation

Apart from the basic task of removing as much infected tissue from the root canals as possible, the mechanical preparation of root canals is focused on providing the canal with a tapered shape. This allows reliable disinfection and tight obturation of the canal. With NiTi instruments, high-quality work can be done in a reliable manner. In the last few years, several power systems for mechanical files have established on the market. The product range provided by Komet comprises both rotary and reciprocating file systems.

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FQ. The way to safe endodontics

Every endodontic treatment is unique, and so is FQ - a well thought-out, innovative system that makes treatments easier, safer and more efficient. An impressive complete solution that translates Komet's experience into a new generation of files. Even difficult teeth can be treated reliably and in a minimally invasive manner with the new rotary file system made of heat-treated nickel-titanium. The innovative variably tapered file core with its double-S cross-section ensures maximum flexibility and effective substance removal. The pre-bendable files impress with their powerful performance. They make work pleasant and give the user a feeling of safety during the treatment. Heat-treated nickel-titanium The higher resistance to cyclic fatigue minimizes the risk of file fracture. Better access to the root canals and lower risk of zipping and ledges, as the file is pre-bendable and very flexible.

- Double-S cross section

Efficient treatment due to the high cutting performance. The narrow core of the file allows for optimum flexibility.

- Concise system

Clearly laid out basic sequence containing a small number of well-matched instruments to save time during root canal treatments. However, the comprehensive, versatile portfolio also allows the compilation of individually adapted file sequences to suit the requirements of the canal anatomy.

Only for Taper .06 Instruments

- Variably tapered file core

The variably tapered file core enlarges the chip space. Infected tissue is transported out of the canal more efficiently and the duration of the treatment is shortened.

Thanks to the heat-treated material, the variably tapered file core is very flexible. The file even follows the anatomy of strongly curved canals, thus increasing the safety of the treatment.

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FQ Opener, taper .08, length 19 mm, sterile packed, for the straight root canal entrance area with small lumen, nickel titanium, heat-treated

FQ03L21
FQ03L25
FQ03L31

			6
Größe · Size	Ø 1/100 mm	015	

Winkelstück · Right-angle (RA)

FQ03L21.204. ... 015

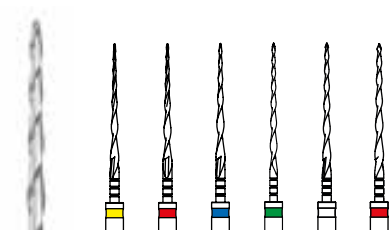
FQ03L25.204. ... 015

FQ03L31.204. ... 015

⌀_{max} 500 min⁻¹/rpm

FQ Glider file, taper .03, sterile packed, for the mechanical establishment of a glide path, nickel titanium
Erstellung eines Gleitpfads, Nickel- Titan

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F360. Anything other than complicated

- Root canal preparation with only 2 files
- Each file is used in rotary mode to its full working length
- Single-use files
- Sterile-packed

Applied intelligence

Prior to using the F360 files, an Opener (e.g. OP10L19) is used to ream the canal access zone to a larger size and remove all infected substance. Subsequently, patency of the canal has to be ensured by means of suitable manual instruments. In most cases, the root canal can now be prepared with just 2 files. The red F360 file in size 025 is first in line, followed by the green F360 file in size 035. The use of the first F360 file in size 025 may be sufficient in case of very narrow canals. On the other hand, the use of additional files in sizes 045 (white) and 055 (red) may be required in case of wide canals.

The F360 files are used in rotary "pecking" motion, i. e. they are moved along the entire working length in a dabbing, pecking manner so that the files are kept moving at all times.

Thanks to their taper of .04, all F360 files are extremely flexible so that they are perfectly able to adapt to different root canal morphologies.

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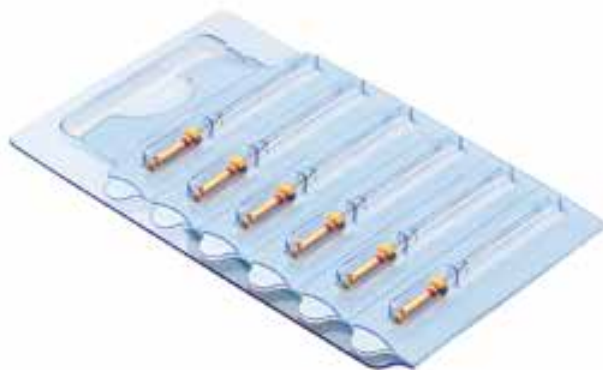
Well thought-out in every detail

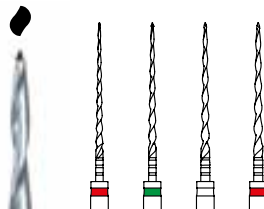
- Manageable file sequence: only 2 files are needed to suit most canal anatomies
- Use in rotary mode in a conventional torque-limited endodontic contraangle or motor
- All files are used with an identical torque of 1.8 Ncm so that there is no need to adjust the torque of the endodontic contra-angle when changing the file
- Simple use according to the single length principle
- Optimum respect of the original canal anatomy
- Excellent cleaning result and quick mechanical preparation thanks to the innovative instrument design: cross-section in the shape of a double S for increased flexibility, large chip spaces to flush out all debris, blade twist with dynamic increase
- Reduced risk of instrument fracture and no risk of cross-contamination as the F360 files are intended for single use only
- No need to clean, disinfect and sterilise instruments. No need to document and control the number of uses as the F360 single-use files are sterile packed
- Flexible files made of NiTi with small taper .04 assure optimal adaptation to nearly all canal anatomies
- The files are also available in the additional sizes 045 and 055 and come in 3 lengths (L21, L25 and L31) to allow for optimal preparation of all canal anatomies

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F360 file, taper .04, single-use file, sterile packed, for canal preparation to the full working length in pecking motion, nickel-titanium

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F360 Gutta-percha points
Colour coded, graduated and radiopaque
Length 28 mm

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Procodile Q. The evolution of flexibility

This evolution makes the endodontic world even safer and easier.

Procodile Q is the first heat-treated reciprocating file with a variably tapered core. For unprecedented flexibility, combined with an incredibly effective performance.

- *Variable core for variable canals: Variably tapered file core for optimum preparation even of curved canals.*
- *Q stands for heat: Heat-treated for outstanding bendability and excellent flexibility*
- *Hungry and fast-cutting: Quick and efficient substance removal thanks to a larger chip space and double-S design.*
- *Safer than ever before: extremely high resistance to cyclic fatigue and file fracture*
- *Procodile Q and you: A perfect match: Suitable for use in all established power systems, perfect in combination with the patented ReFlex® motion.*

Step by Step:

- ➊ Generous preparation of the canal entry zone, for example with the reciprocating Opener OPR08L19.
- ➋ Creation of a glide path, for example with a K-file in size 015. The matching Procodile Q file is selected according to the size of the previously used (manual) file.
Rule:
Skip one ISO file size.
- ➌ Canal preparation with the Procodile Q file.

Obturation with a matching gutta-percha point.

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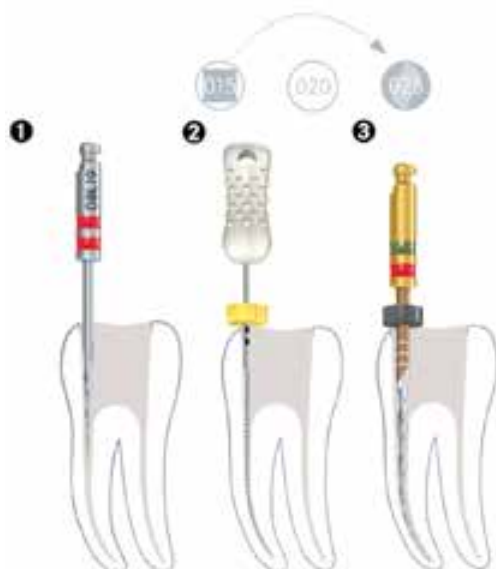
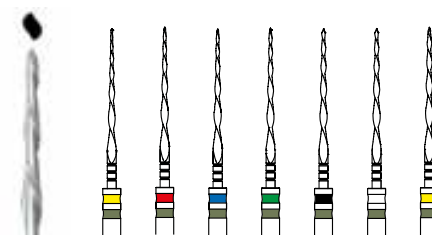
Procodile, size 020-030 with taper .06, size 035-040 with taper .05, size 045-050 with taper .04, single-use file, sterile packed, for the reciprocating preparation of root canals in a "pecking motion" along the entire working length, nickel titanium, heat-treated

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Procodile. Agile, fast and hungry

With its completely revised file design, Procodile is a totally new species in the kingdom of root canal preparation. The innovative, variably tapered file core makes Procodile noticeably more efficient and flexible and therefore ideally suitable for perfect shaping of the root canal and a high-quality preparation. Thanks to its additional green ring on the shank, Procodile can be distinguished easily from other file systems.

- Unrivalled flexibility even in curved root canals thanks to an innovative, variably tapered file core
- Greater efficiency during preparation thanks to an enlarged chip space
- Increased patient safety thanks to greater resistance to cyclic fatigue
- Suitable for use in all common left-cutting reciprocating power units and with the patented ReFlex movement in the EndoPilot

Step by Step

- 1 Generous opening of the canal entrance area for example with the reciprocating Opener OPR08L19
 - 2 Creation of a glide path, for example with a K-file in size 015. The appropriate Procodile file is chosen depending on the size of the previously used (manual) file
- General rule: Skip one ISO size
- 3 Preparation of the root canal with the Procodile file

Obturation with a matching guttapercha point

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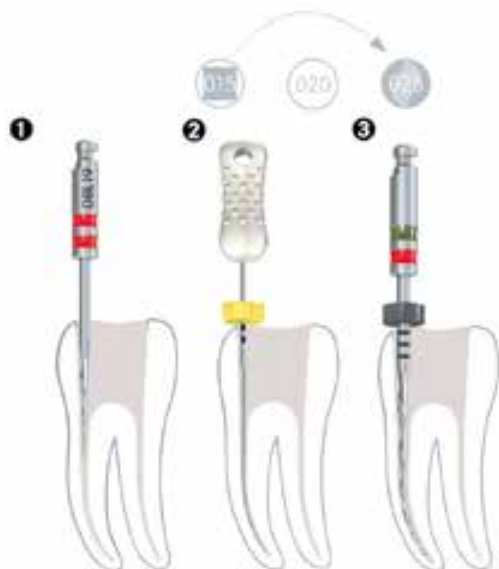
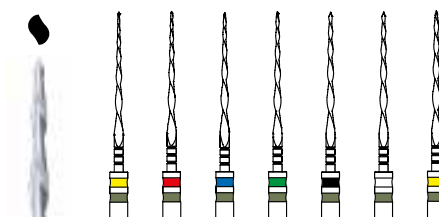
Procodile, size 020-030 with taper .06, size 035-040 with taper .05, size 045-050 with taper .04, single-use file, sterile packed, for the reciprocating preparation of root canals in a “pecking motion” along the entire working length, nickel titanium

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Endo-Antriebe

Endo motors

Special Endo motors are indispensable when it comes guaranteeing a safe mechanical preparation of the root canal. These motors have to meet special requirements, such as reduced speeds and torque control. Komet's endodontic range comprises a torque reduced endodontic contra-angle and a complete endodontic motor capable of combining preparation and electronic determination of the length.

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Endo has many requirements. Here is the solution.

Our latest endodontic motor with integrated apex locator and cordless handpiece promotes your business by making it economical, simple and safe.

The EnDrive makes it easy for you to concentrate fully on your work: Cleaning is less complicated, the battery can be recharged during the treatment and the WLAN updates do themselves at the switch of a button. Thanks to a comprehensive file library you can continue to use your tried and tested favorite file. All this makes working with the EnDrive extremely economical.

Better than easy, it's intuitive: The display is very easy to operate, flexible as it can be used both vertically and horizontally and, thanks to its compact dimensions, you can position or transport the EnDrive as you wish. Comfortable and convenient, cordless and less tiring than manual preparation.

State-of-the-art motor – in complete safety: The EnDrive supports you in your work with a wide range of functions, such as additional feedback on the torque value via the LED ring on the handpiece, perfect visibility thanks to the very small, 360° rotatable contra-angle head and the convenient and hygienic magnetic function for handpiece storage. Included with every EnDrive: our excellent step-by-step advice – we are by your side for as long as you want us to.

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new

ED07.000.

EnDrive - AC/DC Adapter; inklusive 4 Adapter (EU, UK, US, AU)
EnDrive - AC/DC Adapter; including 4 adapters (EU, UK, US, AU)

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Ready, Steady, EnGO! The wireless handpiece with file database.

With Komet's EnGO handpiece, you have a true multi-talent for endodontics in your hands. Whether reciprocating or rotating: EnGO is suited for all common types of movements and offers individual setting options for speed, torque, and angle. The specifications for your most frequently used files can be saved in the unit. Thanks to the pre-installed parameters of all Komet systems, you are all set for your next treatment.

The wireless, ergonomic unit fits perfectly in the hand of both right-handed and left-handed users and makes working comfortable even during longer treatments. Thanks to its 360° rotatable contra-angle, EnGO provides optimal access to the treatment area. In addition, EnGO is particularly safe to use: the small head ensures good visibility, and the integrated apex locator allows you to prepare right up to the apex.

The main arguments:

- Sophisticated Ergonomics
- Optimum View
- Integrated Apex Locator
- Insulated contra-angle
- Individual Settings
- Brushless Motor
- Impressive Performance
- Rotatable all around
- Automatic Movement
- Adjustable Reciprocating Movement
- Easy Handling
- Wireless Charging
- Long-lasting Battery

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EnGO Winkelstück
EnGO contra-angle

ENG003.000.000

EnGO Ladestation
EnGO charging station

ENG004.000.000

EnGO Messkabel für Apex Locator
 EnGO measuring cable for Apex Locator

ENG005.000.000

EnGO AC/DC Adapter für EU
 EnGO AC/DC adapter for EU

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Obturation

Obturation

After the preparation and chemical cleaning of the root canal system, the canals have to be tightly sealed with a suitable obturation to prevent reinfection. Komet provides a comprehensive range of paper and gutta-percha points as well as F360 Fill, a carrier-based filling system. KometBioSeal is a bioceramic sealer and EasySeal is a sealer based on epoxy resin. Root canal sealers are mainly used to fill voids, seal additional root canals and multiple foramina. What's more, endodontic sealers are intended for creating a bond between the filling material and the canal wall and for encasing residual bacteria in the canal.

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Komet BioRepair. The evolution of endodontic treatments

Preserves vitality & stimulates regeneration : The superb vitality-preserving and tissue stimulating properties of Komet BioRepair make it a particularly suitable solution for the treatment of challenging patients.

8 different indications – a single all-in-one solution: BioRepair is ideally suited for the effective treatment of 8 different indications - and each treatment can be carried out with the same solution.

No discoloration despite extensive tooth repair : Another advantage of BioRepair lies in its aesthetic properties, as the color of the treated teeth is not affected even in case of extensive repairs.

User friendly application.: BioRepair Putty is supplied as a premixed solution, which not only ensures easy application, but also enables faster treatments.

- Indication:**
1. Pulpotomy
 2. Root resorption
 3. Apexification
 4. Apexogenesis
 5. Pulp protection
 6. Furcation perforation
 7. Root perforation
 8. Retrofilling

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Komet BioSeal

Komet BioSeal

Komet BioSeal is a new endodontic bioceramic obturation material which perfectly meets all the demands placed on a modern root canal filling material.

Clinical advantages at a glance:

*Bactericidal effect:
KometBioSeal has the ability to effectively kill existing bacteria in the root canal. KometBioSeal owes its excellent antibacterial effect to the initially high pH value during curing.*

*Excellent biocompatibility:
Thanks to its calcium-silicate based material, KometBioSeal has an excellent biocompatibility after having set and, therefore, prevents foreign body reactions by the surrounding tissue. This, in turn, promotes quick healing and reduces post-operative pain.*

*Easy application and short curing time:
KometBioSeal has hydrophilic properties and is ready for use. Unlike conventional sealers based on epoxy resin, KometBioSeal uses the residual moisture from the dentinal tubules for curing. Thanks to the short curing time, combined with the easy and safe application, predictable and reliable obturation results can be achieved using cold filling techniques (for example master cone technique).*

*Optimum sealing ability and perfect adhesive bond:
Highly flowable, KometBioSeal has an optimum sealing ability and creates a perfect adhesive bond between the dentin and gutta-percha. In combination, these two properties prevent the reinfection of the root canal after obturation.*

*Highly radiopaque:
The specific composition of bioceramic sealers is a real challenge in terms of radiopacity. Thanks to its optimized material composition with zirconium oxide particles, KometBioSeal is distinguished by an excellent radiopacity.*

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KometBioSeal – bioceramic sealer
Permanent root filling material based on calcium silicate
Ready-to-use 2 g syringe
Incl. 10 endo tips

BCS1TIPS

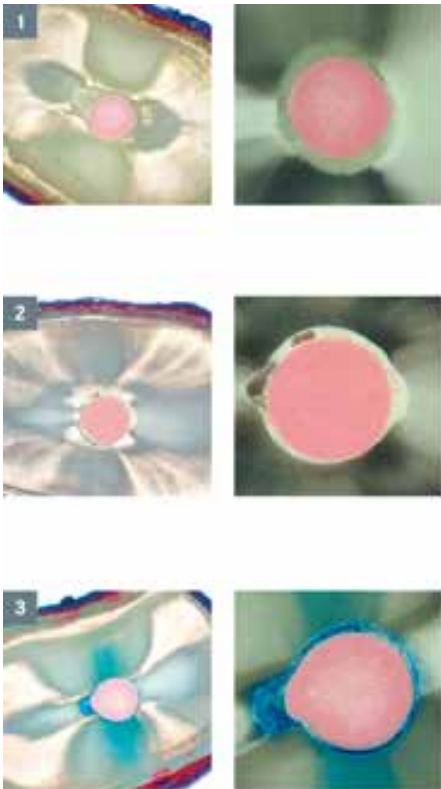
53

		1
--	---	---

BCS1TIPS.000. ...

10 KometBioSeal Endo Tips
10 KometBioSeal endo tips

EasySeal. Root filling material on the basis of epoxy resin

After a successful treatment of the root canal, the root filling is responsible for tightly sealing the canal to prevent reinfection.

A permanent apical seal of the root canal can be achieved with EasySeal, a root filling material based on epoxy resin. EasySeal is particularly suitable for orthograde filling techniques, it is radiopaque and dimensionally stable. EasySeal comes in a double-chamber syringe for easy, safe handling without need for previous mixing. The workable time of EasySeal is 15 minutes. Full setting is reached at 37°C body temperature within 24 hours.

Root filling materials based on epoxy resin, such as EasySeal, are classed as gold standard thanks to their outstanding properties, for example excellent adhesion to dentin, low solubility and high thermo stability.

Leakage test with dye*

1. EasySeal (no dye penetration)
2. Commercially available sealer X (no dye penetration)
3. Commercially available sealer Y (dye penetration)

** Natural tooth, prepared up to ISO 035, filled with different sealers and gutta-percha, immersed in dye penetrant for 48 hours, removed from the dye penetrant, cut in thin slices in intervals of 2–6 mm from the apex and examined for residues of the dye penetrant under a microscope.*

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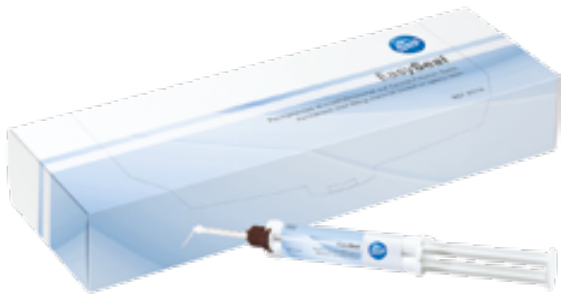
EasySeal
Permanent root filling material on the basis of epoxy polymer
9 g minimix syringe incl. 20 mixing tips and 20 endo tips

9979

55

	1
9979.000. ...	.

20 EasySeal Mixing Tips
20 EasySeal mixing tips

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EnPack & EnFill. Outstanding on their own. Even better as a team.

With our obturation devices for warm, vertical obturation, you have the ideal combination in your hands to complete your endodontic treatments in an economically sound, comfortable and safe manner.

With EnPack and EnFill you are always ready for action. You can get started straight away thanks to the extremely short heating times and work efficiently without interruptions. Powerful rechargeable batteries without annoying cables, ease of operation thanks to the plug-and-play principle and the unique, rotatable displays ensure that you can concentrate fully on your work without worrying about the technology. A quick glance, a quick push of a button, a quick heat-up and you're ready to go.

EnPack and EnFill have everything you need to maintain your usual workflow. Both devices can be used universally, they meet almost any requirement thanks to the 360° rotatable injection needle and their heating tips in nine sizes. The devices allow you to work without fatigue thanks to their ergonomic design and low weight. Increase your treatment efficiency with outstanding, homogeneous results in record time, from the apex to the canal entrance.

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new

PACK004.000.

EnPack Ladestation
EnPack Charging station

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Clinical sequence

1. Prepared, rinsed and dried root canal

2. Determination of the size with the F360 Fill Verifier

3. Choose suitable F360 Fill Obturator and heat up in the oven (green light and + acoustic signal when heated)

4. Apply sealer (e.g. EasySeal 9978)
• with F360 Fill Verifier or paper point
• Apply a very thin coat of sealer onto the canal walls
• Unsuitable for Lentulo

5. Insert obturator into the root canal up to the working length, applying low pressure

6. Remove the protruding core of the obturator and any excess gutta-percha
• Allow gutta-percha to cool for approx. 3-4 minutes
• Break off handle with plastic core by hand
• or with a gutta-percha cutter

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F360 Fill Obturators

Obturator consisting of a plastic core, coated with thermo plastic gutta-percha, to be heated in the F360 Fill Oven to achieve a tight, three-dimensional filling

Universal obturator system, suitable for example for F360 files

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Länge 28 mm

Guttapercha points taper .02

Colour coded, graduated and radiopaque

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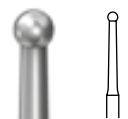
Guttapercha cutter
For cutting guttapercha or the plastic carrier in case of
using a carrier-based obturation system
To be used preferably in the red contra-angle without
cooling agent, applying low contact pressure

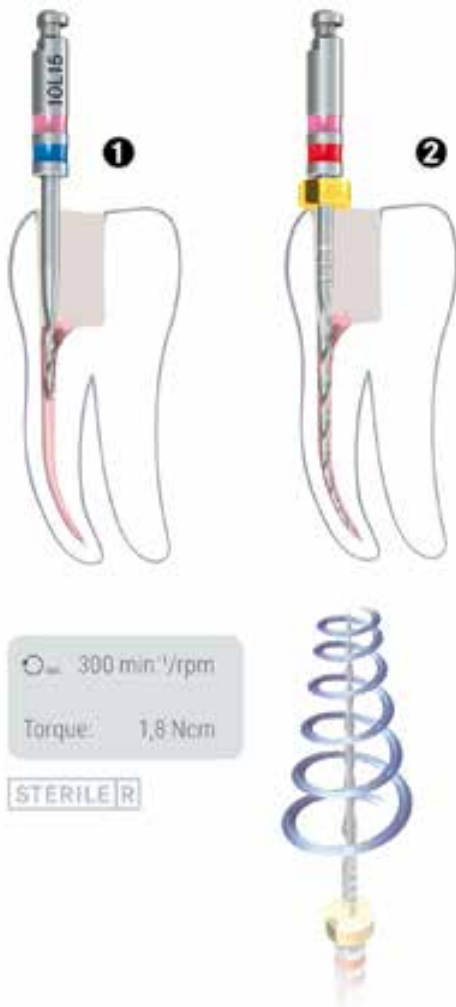
Retreatment

In case of a failed endodontic treatment, for example caused by persistent bacteria or a recolonization of the endodontium, a retreatment is often the only chance to save the tooth. The objectives of an orthograde retreatment are to remove the existing filling material to the extent possible, to access parts of the canal that had not yet been reached and to guarantee a complete re-disinfection of the root canal. The range provided by Komet to this end includes instruments for removing all types of different filling material and a special instrument set for removing fractured files that are stuck inside the root canal.

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Endo ReStart. A new era in retreatment

The goal of every retreatment is to correct an existing endodontic therapy. Thanks to the NiTi retreatment files contained in the Endo ReStart system, this task can now be tackled effectively and under perfect control in just a few steps.

- Endo ReStart is a new NiTi file system for efficient retreatments with just 1-2 instruments
- Thanks to their special blades with dynamic twist, the files of the Endo Restart system ensure effective removal of root fillings
- Thanks to its „safe activity“ instrument tip, we can offer an optimized file to users. The new tip is provided with a tapered, non-cutting end permitting easy penetration of the filling material
- The constant taper of .05 ensures a perfect balance between stability and flexibility

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Endo ReStart Opener, taper .10, length 15 mm, sterile packed, for removing root fillings in the coronal third, nickel-titanium

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Endo ReStart file, taper .05, single-use file, sterile packed, for the removal of root fillings with rotary files to the full working length in pecking motion prior to retreatment, nickel titanium

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Endo ReStart Introductory set 4680 for rotary retreatment of root fillings

Removal of Gutta-percha with the GPR

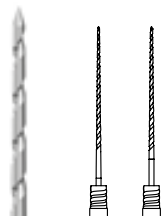
1. Insert the gutta-percha remover GPR between the gutta-percha and the canal wall, rotating the instrument at an optimal speed of $\odot_{\text{opt.}} 2000 \text{ rpm}$ (without irrigation)
2. Once the gutta-percha has softened, increase the speed to the maximum speed of $\odot_{\text{max.}} 4000 \text{ rpm}$ and insert the instrument further towards the apex
3. Pull the gutta-percha remover out of the canal. The plasticized gutta-percha adheres to the instrument

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Gutta-percha remover in taper .02 and taper .04 without cutting edges

Plastification of gutta-percha due to frictional heat caused by rotation, nickel-titanium alloy

In countries other than Germany and Austria the packing unit is 5 instead of 6

Entfernung von F360 Fill Obturatoren

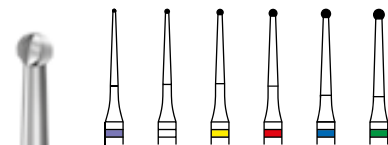
Removal of F360 Fill Obturators

Expose the carrier/plastic core at the coronal part of the tooth, i.e. remove the gutta-percha at the top, for example with an EndoTracer in Größe 010 at an optimum speed of $\odot_{\text{opt.}} 5000 \text{ rpm}$. After that, the carrier can be pulled out with small pincers or special tweezers for removing fragments (ref. 215 or 216).

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H1SML31 Gesamtlänge 31 mm

H1SML34 Gesamtlänge 34 mm

EndoTracer for the preparation of the endodontic access cavity, notably for the preparation of isthmuses

H1SML31 length 31 mm

H1SML34 length 34 mm

Endo Rescue Kit. Removal of instrument fragments

The fracture of an instrument during an endodontic treatment not only causes the dentist enormous stress, it also poses an increased risk of post-endodontic complications to the patient. The removal of such fragments is often extremely difficult and almost impossible to plan.

The Endo Rescue Kit provides a simple and systematic solution, providing access to the opening of the root canal and allowing the removal of the fractured instrument. Once straight access to the fragment has been created by means of a conventional endodontic drill and two Gates burs, two specifically developed instruments greatly simplify a previously complicated procedure:

A centre drill (RKP) exposes the coronal part of the fragment. An extremely fine trepan bur (RKT) is then placed onto the fragment which is seized by the bur and held in place by dentin residues. The fragment is then pulled out of the root in an anticlockwise direction. Thanks to the small diameter of these two instruments, the fragment can be extracted whilst removing the absolute minimum of dentin.

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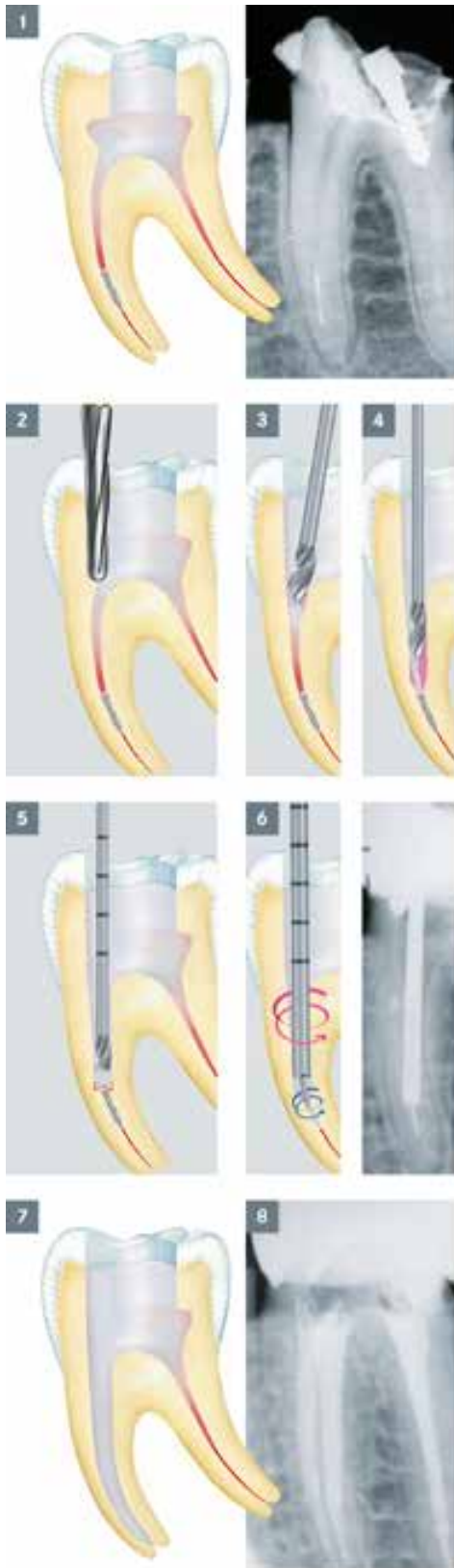
150.155.000

Clinical sequence

1. Initial situation. Presence of a fragment of a fractured instrument in a mesial root.

2. Recreation of the access cavity (H269GK.315.016).

3. The coronal curvature is straightened with axial movements to create direct access to the fragment (G180A.204.110).

4. Preparation of the access right down to the fragment (G180A.204.090).

5. Exposure of the fractured surface by drilling around the fragment (RKP.204.090).

6. The fragment is surrounded and seized. Anti-clockwise rotation. Removal of the fragment which is firmly held in the trepan bur by the residues of dentin (RKT.204.090).

7. The last third can now be correctly prepared.

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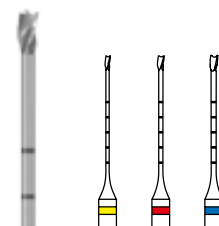
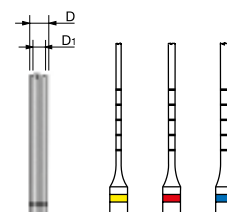
Endo Rescue Trepanbohrer

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Endo Rescue Trepan bur

To be used in anticlockwise rotation

Post Endo

Root posts are intended for the reconstruction of teeth after successfully concluded endodontic treatments. It is their task to ensure permanent retention of the coronal build-up. What's more, the weakening of the root during preparation and load transmission has to be kept to the absolute minimum.

The comprehensive range offered by Komet includes for example the ER system which has been a mile stone in the post endodontic treatment sector for almost 30 years. It comprises tapered, passive root posts that are available in various different material perfectly adapted to each individual indication.

The two short posts, DentinPost X Coated and TitanPost X Coated, made of glass fiber-reinforced composite and pure titanium, perfectly complement the ER system. Thanks to their short shanks of merely 6 mm, the root is hardly weakened at all.

Our line of root posts is completed by the dual curing post cementation and build-up composite DentinBuild Evo.

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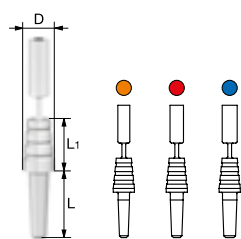
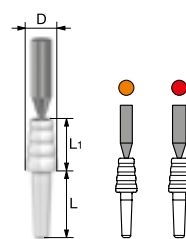
DentinPost X Coated posts with head made of fiber reinforced composite with adhesion enhancing polymer layer, length 6 mm

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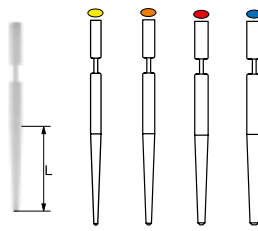
TitanPost X Coated posts with head made of pure titanium with adhesion enhancing polymer layer, length 6 mm

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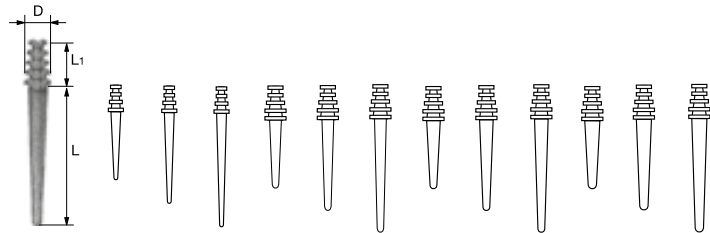
DentinPost Coated made of glass fiber reinforced composite with adhesion enhancing polymer layer

- 48L9
- 48L12
- 48L15
- 228L9
- 228L12
- 228L15
- 49L9
- 49L12
- 49L15
- 50L9
- 50L12
- 50L15

		10	10	10	10	10	10	10	10	10	10	10	10
Größe · Size	Ø 1/100 mm	050	050	050	070	070	070	090	090	090	110	110	110
D	Ø 1/10 mm	20	20	26	28	28	28	28	28	28	28	28	28
L	mm	9	12	15	9	12	15	9	12	15	9	12	15
L ₁	mm	3,5	3,5	3,5	4,5	4,5	4,5	4,5	4,5	4,5	4,5	4,5	4,5

●	48L9.000. ...	050	-	-	-	-	-	-	-	-	-	-	-
●	48L12.000. ...	-	050	-	-	-	-	-	-	-	-	-	-
●	48L15.000. ...	-	-	050	-	-	-	-	-	-	-	-	-
●	228L9.000. ...	-	-	-	070	-	-	-	-	-	-	-	-
●	228L12.000. ...	-	-	-	-	070	-	-	-	-	-	-	-
●	228L15.000. ...	-	-	-	-	-	070	-	-	-	-	-	-
●	49L9.000. ...	-	-	-	-	-	-	090	-	-	-	-	-
●	49L12.000. ...	-	-	-	-	-	-	-	090	-	-	-	-
●	49L15.000. ...	-	-	-	-	-	-	-	-	090	-	-	-
●	50L9.000. ...	-	-	-	-	-	-	-	-	-	110	-	-
●	50L12.000. ...	-	-	-	-	-	-	-	-	-	-	110	-
●	50L15.000. ...	-	-	-	-	-	-	-	-	-	-	-	110

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Composite System

DentinBuild Evo

is a dual curing composite for the cementation of root posts and for core build-ups. DentinBuild Evo is suitable for cementing and building up root posts made of glass fibre reinforced composite, ceramic and titanium.

DentinBond Evo

is a self-etching, dual curing adhesive for use with the DentinBuild composite.

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DentinBuild Evo A2
Dual curing composite for the cementation of root posts and core build-ups.

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DentinBuild Evo Opak white
Dual curing composite for the cementation of root posts and core build-ups.

9972.000.

DentinBond Evo
Selbstätzendes dualhärtendes Adhäsiv
20 x Single Mix Kapseln inkl. 20 Applikationsbürsten
DentinBond Evo
Self-etching, dual curing adhesive
20 x single mix capsules incl. 20 application brushes

9973.000.

DentinBond Evo
Selbstätzendes dualhärtendes Adhäsiv
2x5 ml Flaschen inkl. 50 Applikationsbürsten
DentinBond Evo
Self-etching, dual curing adhesive
2x5 ml bottles incl. 50 application brushes

Sonic tips

The sonic tips from our SonicLine facilitate endodontic treatments with a gentle, conservative touch. Komet not only provides sonic tips for the orthograde preparation of the pulp chamber, but also sonic tips for the retrograde preparation of the root canal as part of an apicoectomy.

Important notes:

The sonic tips made by Komet can not only be used in Komet's sonic hand-piece SF1LM/S, but also

- *in the Scalers made by the co. W&H (Series Proxeo® ZA-55/L/LM/M/LS and Proxeo® ST ZE-55RM/BC, Series Synea® ZA-55/L/LM/M or series Alegro® ST ZE-55RM/BC)*
- *in the SONICflex™ hand-piece provided by the company KaVo (Series 2000N/L/X/LX or series 2003N/L/X/LX)*
- *in the SIROAIR L of the co. Dentsply Sirona*
- *in the sonic hand-pieces Ti-Max, series S970L/KL/SL, made by NSK*

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SonicLine

Sonic tips for orthograde preparation of the pulp chamber and preparation of the cervical third of the root canal as part of an endodontic treatment.

Advantages:

- Quick preparation and removal of old root fillings
- Easier retrieval of root canals
- Enlargement of obliterated canals
- Preparation of straight canal access cavities without weakening the crown
- Useful for removing hard root fillings, cements or posts
- Controlled, gentle preparation without steps and protruding material
- Excellent vision

The sonic tips can be used in the air driven sonic hand piece SF1LM/S (air scaler) and reprocessed with the help of a rinse adapter in a Miele washer/ disinfecter.

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Large bud
Initial preparation of the access cavity and removal of protruding substance

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Small bud
Finishing of the access cavity, minimal refreshing of the dentin layer and removal of residues of root canal fillings

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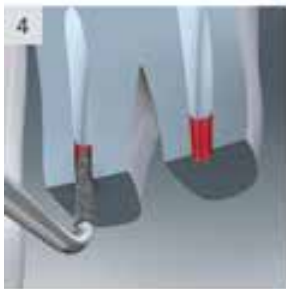
Tapered
Enlargement of long and wide canals, unblocking of fractured instruments, removal of root fillings made of gutta-percha and soft cements

SonicLine

Sonic tips for retrograde endodontic treatments as part of an apicectomy.

Clinical sequence:

Preparative surgical steps

1. Create the access through the jaw bone and work on the bone in the periapical region. Prepare a small bone window with a repositionable lid to ensure a proper preparation of the canal with the endodontic tips for retrograde treatments.

2. Remove the root apex in a right angle to the tooth axis, for example with the tungsten carbide bone cutter Komet H254E

3. Remove any diseased tissue by means of a sharp spoon curette and stop the resulting bleeding.

Retrograde root preparation with SonicLine sonic tips

4. Retrieval of the root canal and exposure of the canal entrance with the sonic tips SF56 (curved to the left) or SF57 (curved to the right)

5. Preparation of the cavity up to a depth of 3 mm with the SF16 (curved to the left) or SF17 (curved to the right) In case of extremely curved root canals, it might be helpful to open and enlarge the canal with the specially shaped sonic tip SF55

6. Preparation of an undercut in order to prevent loss of the canal filling with the tip SF20 (curved to the left) or SF21 (curved to the right)

7. Finally, fill the root with a material suitable for retrograde fillings

Advantages:

- Minimally invasive treatment without need to prepare a large bone window
- Axial work, even in very crowded conditions
- Easier work thanks to double-angled tips
- Slender tips for excellent vision in all jaw regions
- Simplified preparation of undercuts for permanent retention of the retrograde root filling

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Torpedo shaped, extremely curved, Ø 0.7 mm
Opening and enlarging of extremely curved root canals, especially apices strongly inclining in an oral direction

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Sonic handpiece with light and MULTIflex™ connection, incl. tip changer
MULTIflex™ is a trademark of KaVo

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Sonic handpiece with light and Sirona® connection, incl. tip changer
Sirona® is a registered trademark of the company Dentsply Sirona

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Cooling adapter for sonic tips, for external supply of sterile cooling liquid
Stainless steel

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Rinse adapter for reprocessing of the Komet cooling adapter SF1979 or of the ultrasonic tips with internal thread (compatible with EMS and KaVo PIEZOlux™ units) in a Miele washer/disinfector, stainless steel

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Rinse adapter long for reprocessing of the Komet cooling adapter SF1979 or of the ultrasonic tips with internal thread (compatible with EMS and KaVo PIEZOlux™ units) in a Miele washer/disinfector, stainless steel

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Bur block made of stainless steel with 7 holders for sonic or ultrasonic tips and preassembled light blue silicone plugs

Zubehör

Auxiliaries

To support the dentist with helpful tools before, during and after the treatment, Komet offers specially matched accessories and instrument blocks, for example interim blocks for the intermediate storage of endodontic files during the preparation of the root canal or bur blocks suitable for cleaning, disinfection and sterilization of instruments in the autoclave.

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Inserttray

Insert tray

Endodontic instruments can be stored in a clearly arranged treatment tray.

- For 28 endodontic instruments (handpiece and contra-angle)
- PP, 64 x 59 x 19 mm

Sterilization container made of high-performance plastic (PPSU)

- No seals, no maintenance required. More than 2000 sterilization cycles
- Suitable for vacuum steam sterilization
- ePTE filter. The long-lasting ePTE filter remains in the sterilization container during reprocessing (manual or mechanical)
- Transparent material - the contents are visible from the outside
- Two or more containers can be laterally connected

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Sterilisation filter 25 x 61 mm for sterilisation container A8, change after 12 months or after 100 sterilisation cycles, ePTFE, 2 pcs.

Waschbox

Wash box

The wash box is intended for mechanical cleaning and disinfection of instruments in the thermo disinfectant. Following an initial pre-cleaning step, the instruments to be cleaned are placed into the insert trays that are part of the chosen treatment system. The loaded insert tray can then be put into the wash box. It is then cleaned and disinfected in the thermo disinfectant.

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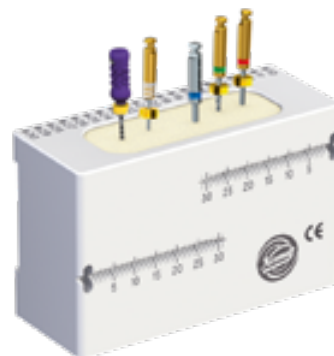
Intermediate support for endodontic instruments with foam inserts (5 pcs)
For the hygienic intermediate storage and cleaning of root canal instruments
during the treatment (without instruments)

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Endo Sequencer, instrument block made of stainless steel and teflon insert with 8 perforations for receiving endodontic instruments Measurement scale with blue knobs to monitor the number of sterilisations. Size 7,3 x 1,3 x 5cm

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9848.000.

Acryl-Übungsblöckchen, 3 St.
Acrylic training block, 3 pcs.

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Komet Italy S.r.l.
Komet Dental Instruments UK Limited