



# Polishers | Kit 43 12A

for polishing composite



This Kit is designed for polishing composite restorations and veneer acrylics. The three-step polishing system with coarse (blue), medium (pink) and fine grit (grey) guarantees an unequalled high-gloss finish on composite surfaces every time.

**Polishing is carried out in three subsequent steps:**

1. The **light blue** polishers are designed for quick and effective pre-polishing of composite fillings.



9400.204.030



9403.204.055



9406.204.100


3. A high-gloss finish is obtained with the fine, **light grey** polishers.



9402.204.030




9405.204.055

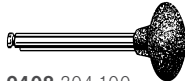


9408.204.100


2. The **light pink** polishers achieve a satin finish.

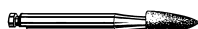


9401.204.030

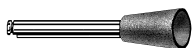

9404.204.055

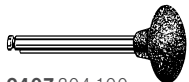

9407.204.100

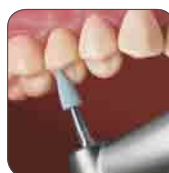


**Recommendations for use:**

- Recommended speed:  
 opt. 5.000 - 6.000 rpm
- Use sufficient coolant (at least 50 ml/min.)
- Apply low contact pressure and work in a circular motion
- Work without polishing paste
- Use alcohol-free detergents and disinfectants suitable for polishers, e.g. Komet DC1 (manual reprocessing) or DCTherm (mechanical reprocessing)
- Suitable for sterilization in the autoclave only



# Polishers | Kit 43 13B


for polishing ceramics

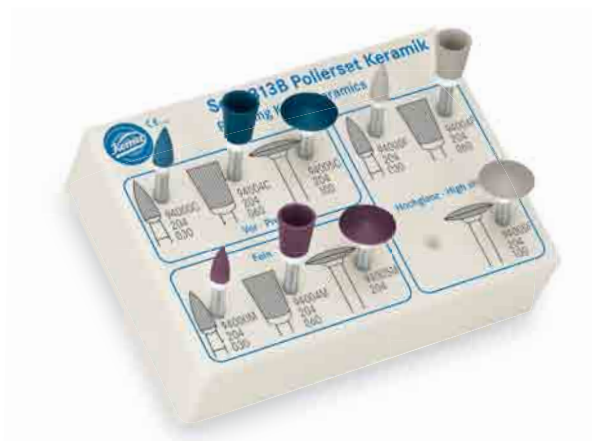

This Kit is designed for polishing ceramic restorations. The amazing surface quality obtained by these diamond interspersed polishers for pre-polishing, polishing and high-gloss polishing obviates the need for another final firing step after corrections have been made.

**Polishing is carried out in three subsequent steps:**

1. The **dark blue** polishers are designed for quick and effective pre-polishing of ceramics.

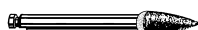

**94000C.204.030**




**94004C.204.060**



**94005C.204.100**

3. A high-gloss finish without need for an additional final firing step is obtained with the wear-resistant, high-performance **grey** polishers.

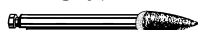

**94000F.204.030**



**94004F.204.060**



**94005F.204.100**


2. The **dark pink** polishers achieve a satin finish.



**94000M.204.030**



**94004M.204.055**

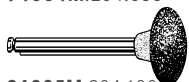

**94005M.204.100**



**Recommendations for use:**

- Recommended speed:  $\text{opt. } 5.000 - 6.000 \text{ rpm}$
- Use sufficient coolant (at least 50 ml/min.)
- Apply low contact pressure and work in a circular motion
- Work without polishing paste
- Use alcohol-free detergents and disinfectants suitable for polishers, e.g. Komet DC1 (manual reprocessing) or DCTherm (mechanical reprocessing)
- Suitable for sterilization in the autoclave only

