



# Grinders & Polishers | **ETNA**

---



## Premium Tools for Perfectionists. Crafted for Complete Ceramic Excellence.

Monolithic restorations are the gold standard in prosthetic dentistry, where both the natural appearance and performance of the restoration are influenced by its shape and surface quality. With the growing diversity of modern all-ceramic materials, the demands on dental professionals have never been higher, requiring not only precision but also flexibility in processing. The new Komet ETNA grinders and matching polishers are designed to meet these high expectations, especially for dental laboratories that strive for perfection.

With a focus on durability, precision, and efficiency, ETNA grinders are characterized by a harder bond and high diamond grit concentration, enhanced edge stability, and superior material removal. These tools prepare surfaces that can be polished directly, thus eliminating the need for extra steps.

ETNA polishers complement the grinders perfectly as they are made of a softer material and are provided with a special diamond grit that ensures a smooth, effortless workflow.

Delivering a high-gloss finish in just two steps, they also offer exceptional durability for consistent performance over a long period of time. Their versatility allows both grinders and polishers to adapt seamlessly to all ceramic shapes and sizes, including zirconia.

Whether in the lab or the clinic, ETNA empowers professionals with premium tools for premium results. Trust Komet for high-quality, holistic solutions that simplify your workflow while delivering exceptional outcomes.



#### Advantages of ETNA Grinders:

- Exceptionally long service life
- Effective on all types of ceramic, including zirconia
- High concentration of diamond grains in a stable special bond
- Excellent stability of the shape and edges
- Fine surface finish - ideal for subsequent polishing

#### Recommendations for use:

-   $\text{opt. } 10\,000 - 12\,000 \text{ rpm}$
- Work with little pressure

#### ETNA Grinders:



DCB1BA.104.025



DCB2BA.104.048



DCB3BA.104.040




DCB4BA.104.120



DCB5BA.104.220



DCB6BA.104.120

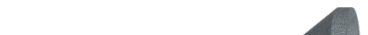

DCB7BA.104.220




DCB8BA.104.035




DCB9BA.104.050



DCB10BA.104.040

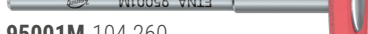


DCB11BA.104.060

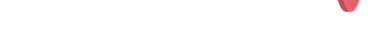

DCB12BA.104.035



DCB13BA.104.150

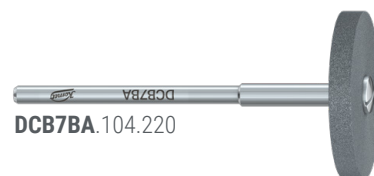

DCB7BA.104.220



DCB8BA.104.035



DCB9BA.104.050



DCB10BA.104.040



DCB11BA.104.060



DCB12BA.104.035




DCB13BA.104.150



#### Advantages of ETNA Polishers:

- Special diamond grit:  
Creates a flawless surface finish with minimum pressure.
- Perfect for direct polishing after working with the ETNA Grinders.
- Two-step process:  
Achieves a high-gloss finish in just two steps.
- Softer material for effortless polishing.
- Flexible application:  
Perfect for various ceramic shapes and sizes.

#### Recommendations for use:

- 1. Step: Prepolishing (Red)
- 2. Step: High-shine polishing (Grey)
-   $\text{opt. } 6\,000 \text{ rpm}$

#### ETNA Polishers:



95001M.104.260




95001F.104.260



95002M.104.150



95002F.104.150

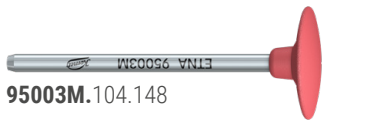

95003M.104.148



95003F.104.148



95004M.104.170



95004F.104.170



95005M.104.200



95005F.104.200



95006M.104.055



95006F.104.055